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©2008 Siemens Healthcare Diagnostics Inc.

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Dimension Vista® System

UCFP CAL

Urinary/Cerebrospinal Fluid Protein Calibrator

Intended Use
The UCFP CAL is an *in vitro* diagnostic product for the calibration of the Urinary/Cerebrospinal Fluid Protein (UCFP) method on the Dimension Vista® System.

Reagents
Contents: 3 vials: (Calibrator A, 1.5 mL per vial)
UCFP CAL is an aqueous product containing dilute human serum.
Assigned Constituent Values: Refer to the enclosed Table of Assigned Values.

Constituent	Traceability
UCFP	NIST SRM® 927

a. NIST-SRM: National Institute of Standards and Technology - Standard Reference Material

Precautions: Contains sodium azide (<0.1%) as a preservative. Sodium azide can react with copper or lead pipes in drain lines to form explosive compounds. Dispose of properly in accordance with local regulations.
Contains human source material. Each donor unit used in the preparation of this product was tested by FDA-approved methods for the presence of antibodies to Human Immunodeficiency Virus Type 1 (HIV-1) and Type 2 (HIV-2), as well as for Hepatitis B Surface Antigen (HBsAg) and antibody to Hepatitis C Virus (HCV), and found to be negative (not repeatedly reactive). Because no testing can offer complete assurance that these or other infectious agents are absent, this material should be handled using good laboratory practice to avoid skin contact and ingestion.
For *in vitro* diagnostic use.

Reagent Preparation: UCFP CAL is ready for use. No preparation is required.
Materials Provided: UCFP CAL, Cat. No. KC260
Materials Required But Not Provided: None
Store at: 2 – 8 °C
Stability: **Unopened Product** - See Expiration Date.
Opened Product:
- Once the stopper of the vial is punctured, assigned values are stable for 7 days when stored on board the Dimension Vista® System.
- Once cap is removed, assigned values are stable for 31 days when recapped immediately after use and stored at 2 – 8 °C. Do not use this vial on board the instrument.

Calibration Procedure
Refer to the Calibration and Quality Control section in the Operator's Guide for the Dimension Vista® System.

Symbols Key: See adjacent panel.

Dimension Vista® System is a trademark of Siemens Healthcare Diagnostics Inc.
©2008 Siemens Healthcare Diagnostics Inc.
All rights reserved.